DE

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Indication

The VBM Pelvic Sling helps to fix and stabilize pelvis fractures by EMS personnel in the prehospital setting.

Fractures of the pelvis are very painful and the inner bleeding resulting thereof might lead to a life-threatening condition for the patient. Each movement between body and lower extremities might increase the bleeding and can result in damage to the inner organs.


The Pelvic Sling exerts a lateral pressure to the pelvis to push it back to its original position. Stabilization should be achieved soonest, if possible prior to evacuation of the patient.

Contraindication



For unconscious patients only use the Pelvic Sling if the airway has been secured adequately. In case of pregnancy the Pelvic Sling should not be used.

Use



1. Remove all objects from the patients' pockets in the area of the pelvis.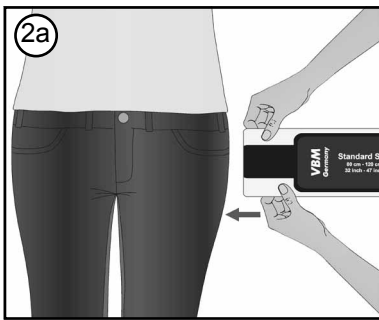

2. Choose the correct size and slide the Pelvic Sling with the help of the Slide Splint under the patient into position (blue side facing up) (2a). It might help to start level with the waist or the thigh of the patient. Afterwards remove the Slide Splint and position the Pelvic Sling so that the upper rim is placed at the upper part of the iliac crest. The cuff markings should be equally visible on the left and right side (2b).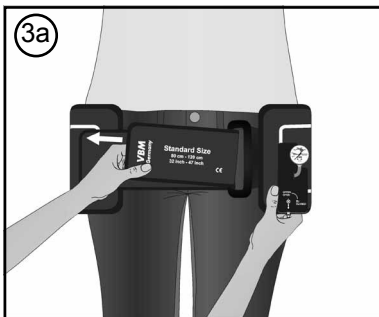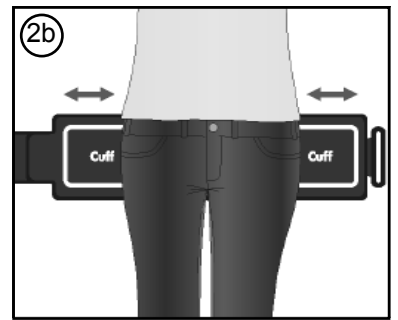

3. Close the Pelvic Sling similar to a belt by sliding the fixation tape through the buckle and pulling it slightly (3a). If necessary, adjust the length of the fixation tape according to the patient (3b). The Pelvic Sling should be fitted tightly to the patients' body but without changing the position of the pelvis.

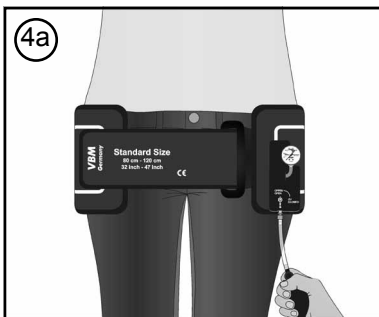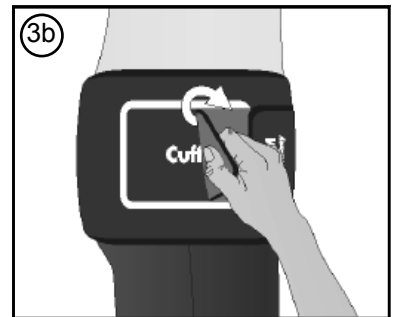4. Both cuffs are inflated simultaneously with the hand inflator (4a). Then close the stop-cock. The Pelvic Sling is well tolerated by the awake patient and inflated in accordance with the patient. For non-responsive (unconscious) patients it is recommended to inflate to a pressure of 70-100 mmHg, if necessary to max. 140 mmHg.

5. If necessary, fix the knee with the Slide Splint (5).

6. The Pelvic Sling is radiolucent and can remain on the patient for CT or MRI **(Without hand inflator. Make sure the stop-cock is closed!)**.

Warnings

- After cuff inflation close the stop-cock and control proper cuff inflation during transport (pressure control at the manometer).
- Only remove the Pelvic Sling after finished patient transport to the hospital, once a surgical stabilization of the pelvic ring can be guaranteed.
- Do not use the Pelvic Sling if there are visible damages or contamination.

The Pelvic Sling is a non sterile medical aid for single use and must not be reused. Further recommendations for our single use products as well as recommended storage conditions are available at www.vbm-medical.de in the download section.

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