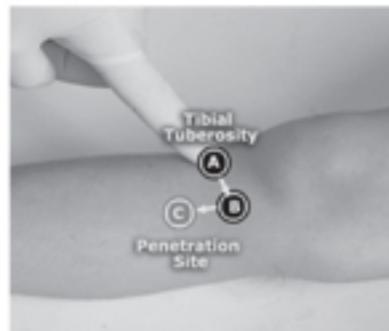
## E N G L I S H

**1**

Find & Mark penetration site.

**Primary Site for Intraosseous Injection Proximal Tibia.**

**Infants 0-6 years old:** 1cm medially and 1cm distally to the tibial tuberosity.

**Pediatrics 6-12 years old:** 1-2cm medially and 1-2cm distally to the tibial tuberosity.

**\*Secondary Sites for Intraosseous Injection - Medial Malleolus**

1-2cm proximally to the base of the medial malleolus (4-5cm above the tip of the medial malleolus).

**2**

Choose the desired depth of penetration.

**3**

Position the BIG with one hand to the site and pull out the Safety Latch with the other hand.

**4**

Trigger the BIG at 90° to the surface.

**5**


Remove the BIG.

**6**

Pull out the stylet trocar.

## Instrument Drawings:

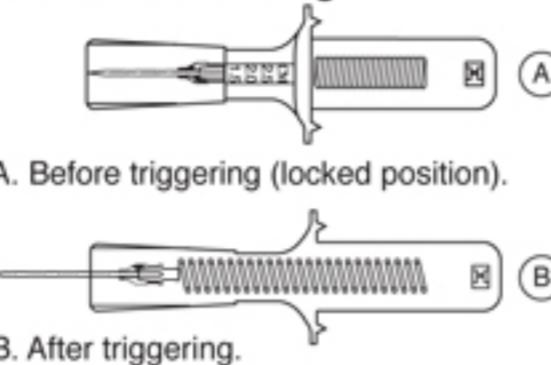Recommended Penetration Depth:  
Pediatric (Red 18G)


	Proximal Tibia	Medial Malleolus
Infants 0 to 3 years of age	0.5 - 0.7 cm	0.75 - 1.0 cm
Pediatrics 3 to 6 years of age	1.0 - 1.5 cm	
Pediatrics 6 to 12 years of age	1.5cm	1.0 cm

**7**


Fix the cannula with the Safety Latch.

**8**

Connect infusion set with a stopcock to a standard IV set (2.5ml, 5ml or 10ml syringe may be used). If necessary, 2-5cc of bone marrow can be aspirated into a heparin-coated syringe for laboratory sampling. For conscious patients consider flush of local anesthesia.

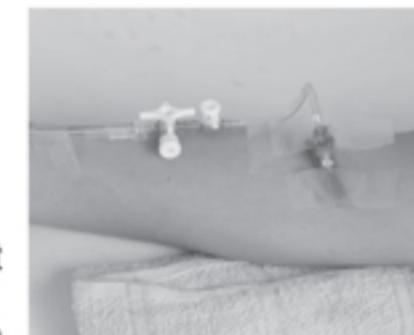**9**

Adjust flow or inject drugs and fluids. Optimal flow maintains by high pressure up to 300 mmHg to the infusion bag.

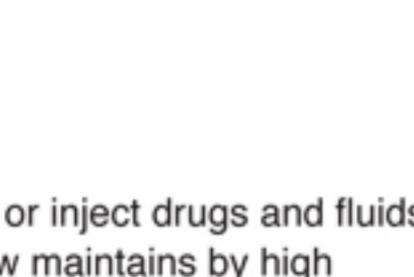\*CE approved for all ages and injection sites.

**Warning:** Use of the B.I.G. is restricted to trained and skilled medics, nurses and paramedics.

**Caution:** The B.I.G. device is for single use only.

**Contraindications:** Skin infection, tumour, Osteogenesis imperfecta, Osteopetrosis, A second attempt following previous I.O. insertion/failure on the same bone, Previous orthopedic procedures near insertion site and/or fracture at the insertion site.

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WaisMed Ltd.
Data Southern Enterprises Ltd.