### ENGLISH

1

Find & Mark penetration site.

#### Primary Site for Intraosseous Injection Proximal Tibia.

2cm medially and 1cm proximally to the tibial tuberosity.

#### Secondary Sites for Intraosseous Injection

**Medial Malleolus** - 1-2cm proximally to the base of the medial malleolus (4-5cm above the tip of the medial malleolus).

**Distal Radius** - Posterior-distal metaphysis of the radius (opposite the radial pulse area).

**Humerous** - Anterior head of the humerus.

2

Choose the desired depth of penetration.

3

Position the BIG with one hand to the site and pull out the Safety Latch with the other hand.

4

Trigger the BIG at 90° to the surface.

5

Remove the BIG.

6

Pull out the stylet trocar.

7

Fix the cannula with the Safety Latch.

8

Connect infusion set with a stopcock to a standard IV set (2.5ml, 5ml or 10ml syringe may be used).

If necessary, 2-5cc of bone marrow can be aspirated into a heparin-coated syringe for laboratory sampling.

For conscious patients consider flush of local anesthesia.


\* F.D.A. approved for adults in proximal tibia. CE approved for all ages and injection sites.

**Warning:** Use of the B.I.G. is restricted to trained and skilled medics, nurses and paramedics.


**Caution:** The B.I.G. device is for single use only.

**Contraindications:** Skin infection, tumour, Osteogenesis imperfecta, Osteopetrosis, A second attempt following previous I.O. insertion/failure on the same bone, Previous orthopedic procedures near insertion site and/or fracture at the insertion site.

#### Instrument Drawings: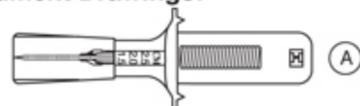

A. Before triggering (locked position).

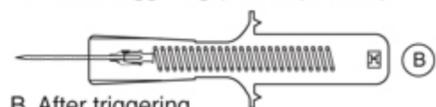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WaisMed Ltd.
Data Southern Enterprises Ltd.