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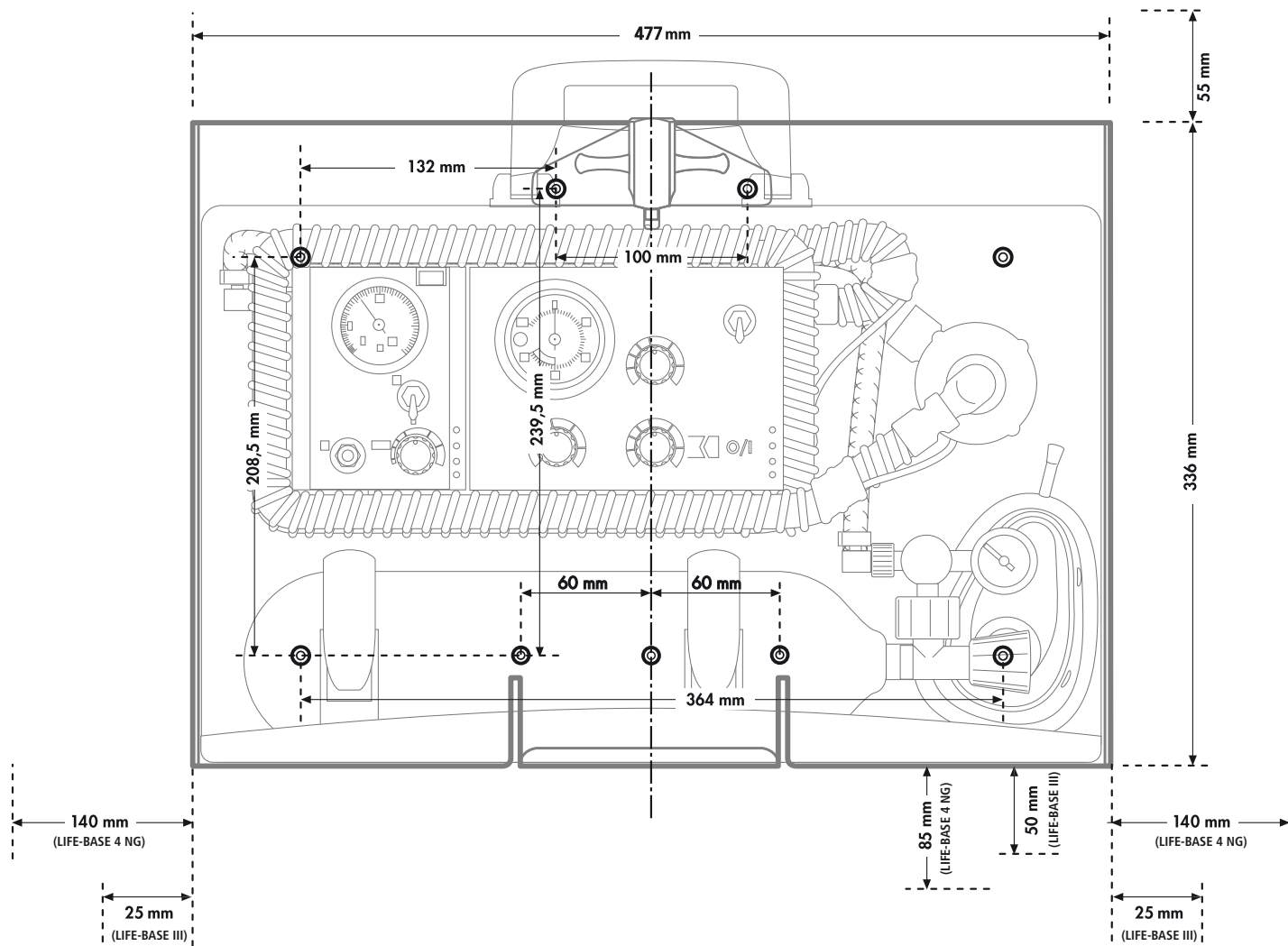
Warnung!

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Fitting the wall bracket

Warning!

- The wall mounting BASE STATION 3 NG may only be used for hanging up carrying platforms LIFE-BASE III and LIFE-BASE 4 NG. The weight and dimensions of these carrying platforms are exactly matched to the wall mounting, so use of other carrying platforms is not permitted.
- Hanging up third-party products or adapters from other manufacturers is not permitted, as this does not comply with the use in accordance with the purpose of this wall mounting.

Fitting steps

- Look for a suitable place to attach the wall mounting, somewhere **plane** and **vertical**.
- Mark all 9 bores at the mounting point.
- Drill the holes using a 4.2 mm drill bit.

- Insert nine M5 threads in the surface for attachment.
- Screw on the wall mounting using the 9 screws enclosed. The carrying platform must be mounted horizontally.
- Connect the charger interface to the 12 V on-board electricity supply of the emergency vehicle using the plug connector for vehicles.

Note:

- The surface for attachment must be sufficiently firm to hold the wall mounting and the carrying platform securely to EN 1789 (ambulances) and/or RTCA DO 160E.
- The charger interface connection to the 12 V on-board electricity supply must be fused with an 8 A fuse.
- The wall mounting has been tested exclusively with M5 mounting screws to EN ISO 4762 (strength 8.8). If other screws are used, ensure that they are of equivalent or higher strength.
- The fitter is responsible for the fitting of the wall mounting.

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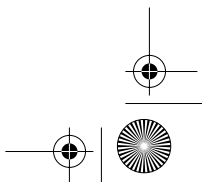
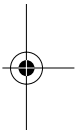
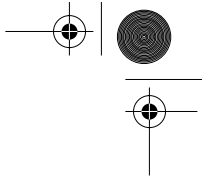
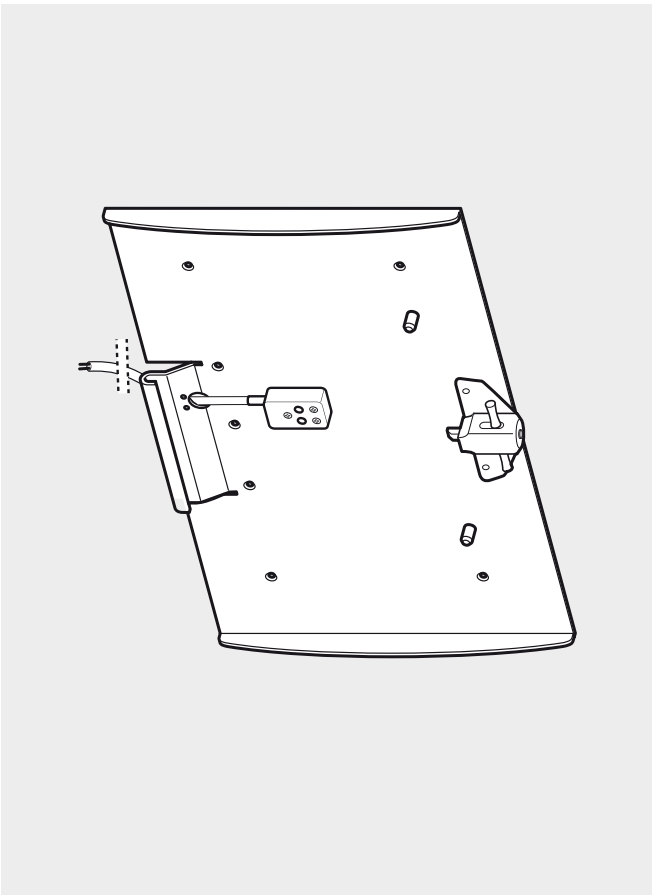
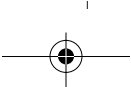
Wandhalterung für LIFE-BASE III und LIFE-BASE 4 NG
Wall mounting for LIFE-BASE III and LIFE-BASE 4 NG
Support mural pour LIFE-BASE III et LIFE-BASE 4 NG

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Geräte für Medizin GmbH+Co. KG