DE GB FR

BASE STATION 1 NG

Wandhalterung / Wall bracket / Support mural

BASE STATION 1 NG

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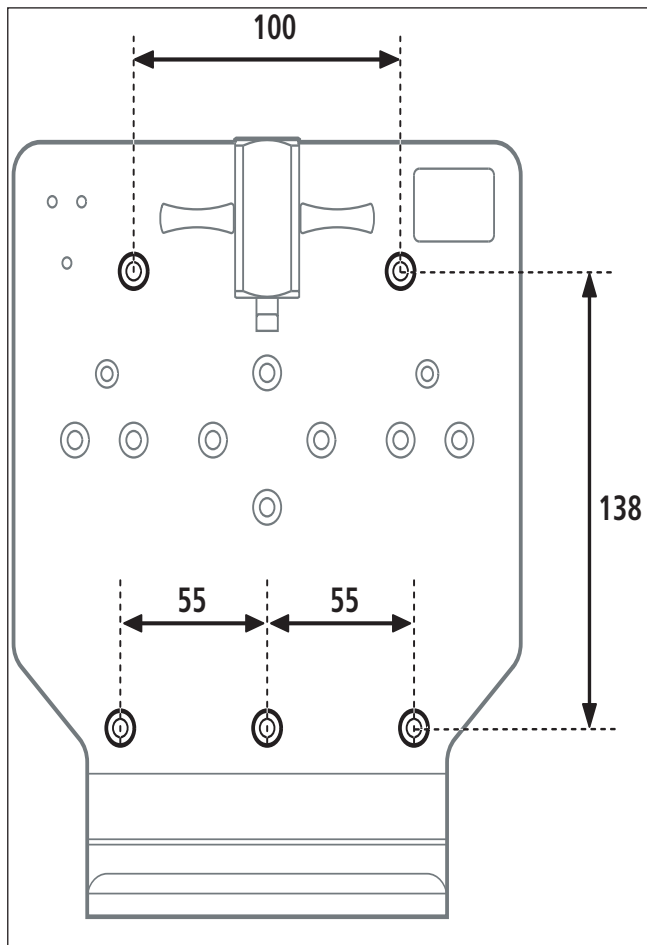
These fitting instructions describe how to fit the BASE-STATION 1 NG wall bracket. The wall bracket is an accessory for the LIFE-BASE 1 NG carrying system. Read the operating instructions for the LIFE-BASE 1 NG carrying system (WM 67061) as they are a component of these fitting instructions. The scope of supply includes the parts required to attach the wall bracket to the wall. Using additional attachment sets, you can also fit the wall bracket to a standard hospital rail (attachment set WM 15805) or to an ordinary rail (attachment set WM 15806). The relevant fitting instructions describe how to assemble the attachment sets.

Safety information

Warning!

- Only the LIFE-BASE 1 NG carrying system may be suspended from the wall bracket. The weight and dimensions of this carrying system are exactly matched to the wall bracket. Other carrying systems may not be used.
- Suspending third-party products or other makes of adapter is not permitted as this does not comply with the use in accordance with purpose of this wall bracket. Refer to the instructions for use of the LIFE-BASE 1 NG carrying system.

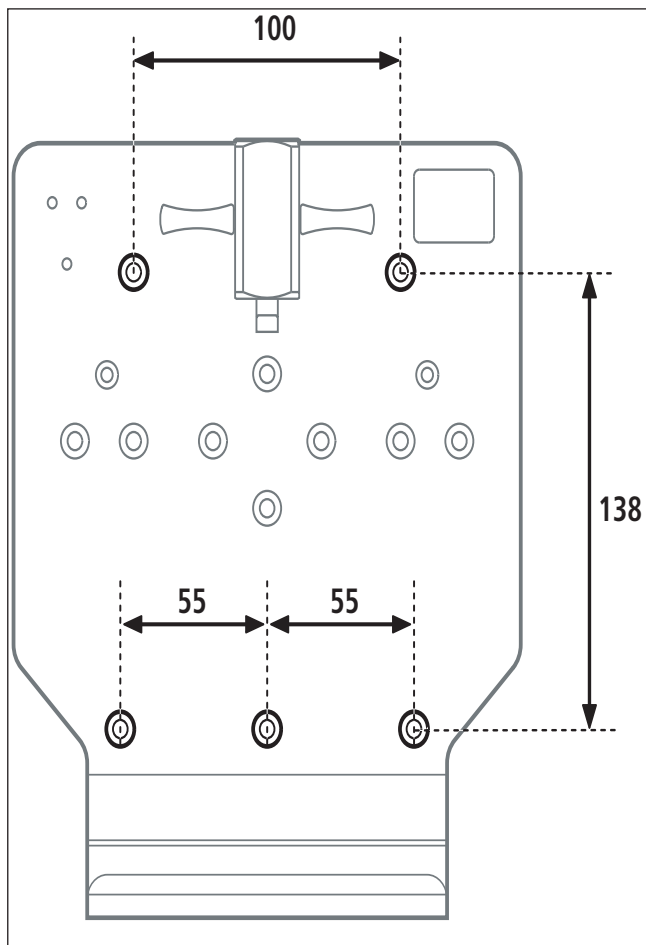
Note

- The surface for attachment must be sufficiently strong to hold the wall bracket and the carrying system securely in accordance with DIN EN 1789 (ambulances) and/or RTCA DO 160E.
- The charger interface connection (optional) to the 12V on-board electricity supply must be fused with an 8A fuse.
- The wall bracket has been tested exclusively with M5 mounting screws to EN ISO 4762 (strength 8.8). If other screws are used, ensure that they are of equivalent or greater strength.
- The fitter is responsible for the fitting of the wall bracket.

Fitting to a wall

1. Look for a suitable place to mount the system, somewhere **flat** and **vertical**.
2. Mark 5 boreholes where you want to fix the system, as indicated in the drawing on the next page.

Note: You do not need all the bores in the wall bracket to attach it to a wall. The drawing on the next page shows you which bores you have to use and how they are spaced.
3. Drill the holes using a Ø 4.2 mm bit.
4. Insert 5 M5 threads in the substrate.
5. Screw on the wall bracket using the screws supplied. The carrying system must be fitted horizontally.
6. If present, connect the charger interface to the 12V on-board electricity supply of the emergency vehicle using the connector for vehicles.

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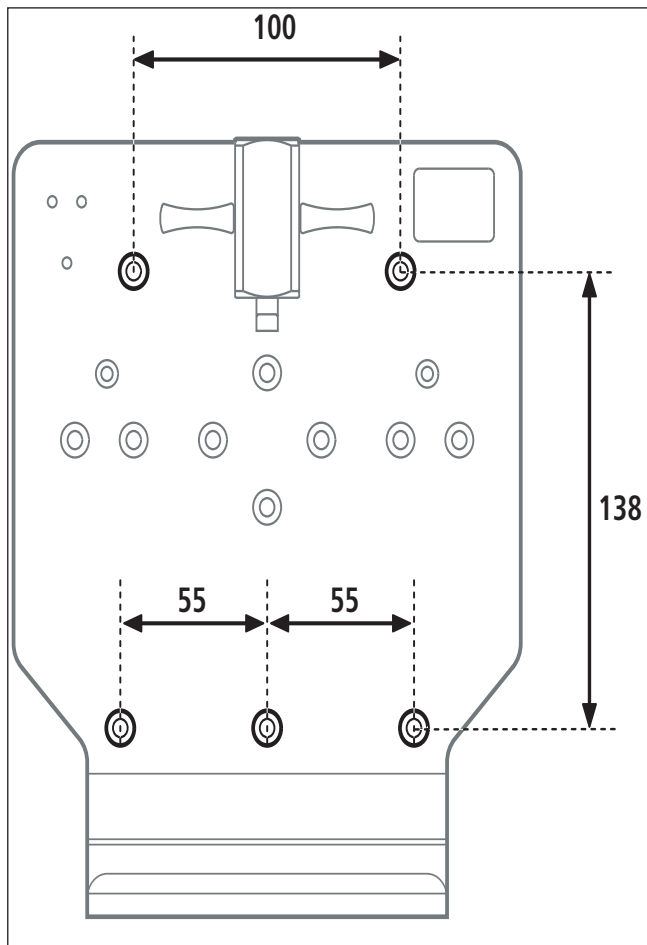
**Center for
Production, Logistics, Service**

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Geräte für Medizin GmbH+Co. KG**